

Internal Organization Correspondence

RF Goodrich

FROM E. B. Katzenmeyer, Jr.	FIELD POINT OR DEPT. & BLDG. NO. Akron, D/0020, B/5-H	25-1612	DATE THIS LETTER August 23, 1976
TO O. S. Kurtz	FIELD POINT OR DEPT. & BLDG. NO. Brecksville R & D Center, D/8506		

In answer to your request for preservative information for urine collection, please amend paragraph 4 of instructions dated July 29, 1976, to read:

Urine with a specific gravity of less than 1.010 is not satisfactory for examination and will have to be recollected. A minimum of 100 mls is to be collected for lead determination. Add 1/2 ml (0.5 ml) concentrated hydrochloric acid per 100 mls of urine as a preservative.

O. S. Kurtz
O. Sue Kurtz

jb

cc: P. M. Zakriski
R & D Files

BFG-I- 504